

mobiskins™

Application Instructions


READ AND FOLLOW ALL INSTRUCTIONS BEFORE APPLYING!

(Especially if you are not inclined to read directions.)

Applying skins is easy. But it is possible to damage them if instructions are not followed.

Description. Mobiskins are a fun way to enhance your Mobilegs™. The printed skins are laminated to protect against scuffs and scratches. Mobiskins are self-adhesive backed on a peel-off carrier sheet. Although the adhesive is relatively low tack and the skins repositionable, it is best to place them as accurately as possible to prevent distortion of the skins. Once placed, they will stay in position. They can be easily removed at any time, but cannot be re-applied.

Caution! Care should always be taken when applying Mobiskins to avoid adhesive-to-adhesive contact or distortion of the material (by stretching) when repositioning.

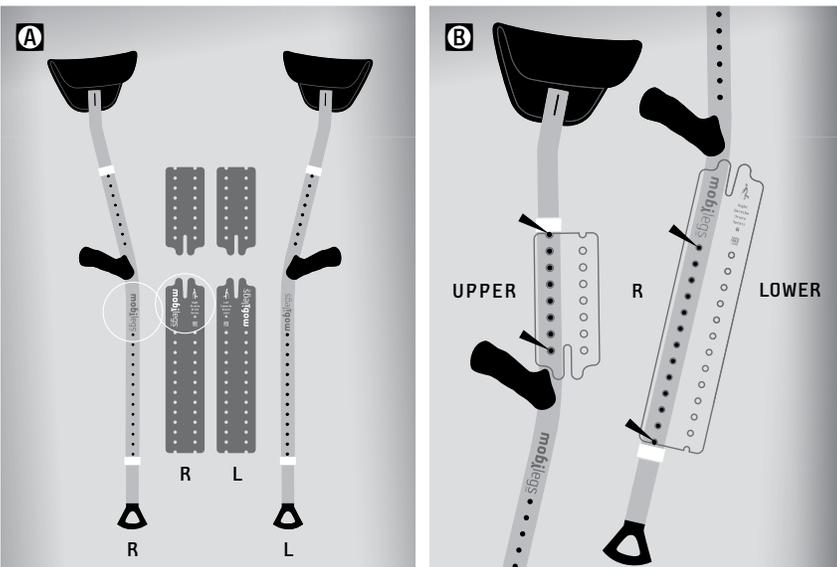

A

Prepare. Work in a clean, well-lit environment. Lay your Mobilegs™ and Mobiskins out on a table. Your Mobilegs™ are right and left-handed. So are the skins. Pair the correct Mobiskins with the corresponding leg. The left and right skins will be labeled in the same way as your Mobilegs™ are. The upper skins are usually identical for right and left sides. However, some designs may vary, in which case, take care to match them. Lay the paired skins out on the table as they will appear on the Mobilegs™.

Before you start.

Be sure the surface of your Mobilegs™ is clean, dry and dust-free.

B

Hold it! Before peeling the backer, test wrap a skin around one of your Mobilegs™. You will see that the holes on the skin correspond with the holes on your Mobilegs™.

Begin with the right side Mobileg™

Lay the right Mobileg™ with the branding side up and handgrip oriented toward you.

C

Start with the upper section, the shorter of the two right skin pieces. It will give you the hang of mounting skins and simplify application of the lower leg section. Peel the long side of the top section back about halfway.

D

Carefully align the top hole on the leg with the notched hole at the top of the skin.

E

Gently contact only the area immediately around the top hole and then align the bottom hole of the skin with the bottom hole of the leg (hole nearest the handgrip).

F

When the holes are in perfect alignment push the skin down lightly along the holes. Then lightly tack down the remaining exposed edge beginning in the middle and working toward the ends.

G

Again, beginning in the middle of the skin, tack it down working slowly toward the ends and rolling down the label as it is being peeled back. Make sure the label is going down flat. If not, stop. Uniformly pull back the label and reapply.

H

Pay extra attention to the two lobes that extend toward the handgrip on the top and bottom skins. Tack slowly and they should go down flat. If creases form, gently pull back a large section of the skin. Do it slowly and uniformly to avoid stretching the material. Do not pull back a corner only. Then reapply. Your patience will be rewarded.

Dealing with wrinkles. If the skins are applied according to instructions, wrinkles will be minimal. To unwrinkle, gently lift the edges near a wrinkle. Slowly and deliberately pull the skin material back away from the wrinkle. Always lift a large section of skin, to avoid distorting an area of the skin.

Repeat. Follow the same procedure until all skin sections are applied.

Job well done! Wear your skins with pride.

